


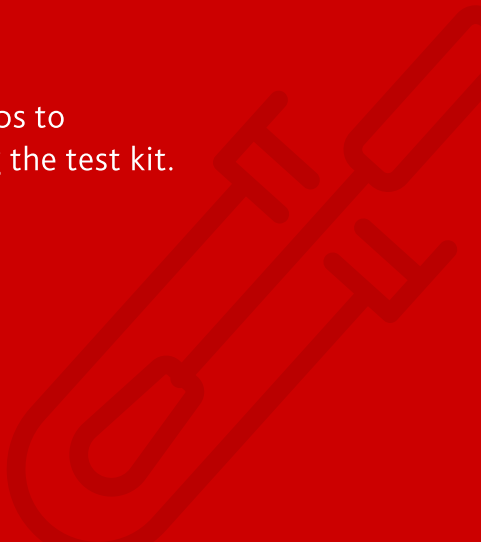

HOW TO USE YOUR

# COVID-19 RAPID ANTIGEN SCREENING KIT

Follow these steps to  
self-screen using the test kit.



# 1



Sanitize hands and lower mask to just below nose.

# 2




Insert swab into one nostril, 2 cm in depth, and rotate around nostril 5 times. Repeat in opposite nostril.

# 3



Open tube, insert swab, & circulate the swab in liquid 5 times.

# 4

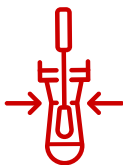

Use one hand to squeeze tube & pull up swab with other hand, bringing swab through squeezed tube.

# 5



Discard TOP of the swab, keeping lower part of swab in tube. Cap tube.

# 6



Unscrew bottom white cap, & apply 5 drops of fluid onto cartridge.

# 7



Close nozzle, put tube on desk, and press "start" on a 15 min timer.

# 8



Sanitize hands. Wait 15 minutes. Interpret result:

**1 line at "C" = negative**

**2 lines at both "C" and "T" = positive**

(regardless of how faint either of the lines are)

**1 line at "T" or no lines = invalid**

Follow-up with your Brock contact to discuss next steps.



Visit [brocku.ca/coronavirus/antigen](https://brocku.ca/coronavirus/antigen)  
or scan the QR code to view these instructions  
online and learn more about the rapid antigen  
screening program.

For the latest updates on Brock's response to COVID-19,  
visit [brocku.ca/coronavirus](https://brocku.ca/coronavirus)