



HOW TO INTERPRET YOUR SCREEN RESULTS

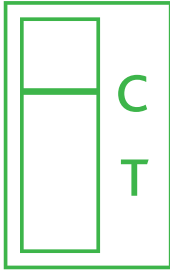

NEGATIVE

COVID-19 is not currently detected

- The presence of 1 line to the left of the “C” with no line at “T”, indicates a presumptive negative result
- Of note, there is a chance that this result may be a false negative, so continue to follow all Public Health protocols (ie. masking, physical distancing, hand hygiene, etc.)

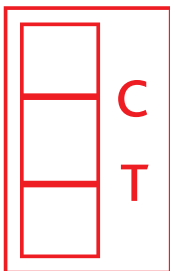


PRESUMPTIVE POSITIVE

COVID-19 is detected

- The presence of 2 lines to the left of both the “C” and “T”
- A confirmatory PCR laboratory-based test is required within 24 hours

Next steps:

- Please begin to self isolate and follow Public Health protocols and guidance
- Book an appointment for PCR testing at your nearest COVID-19 testing centre
- Call your manager to inform them of your result

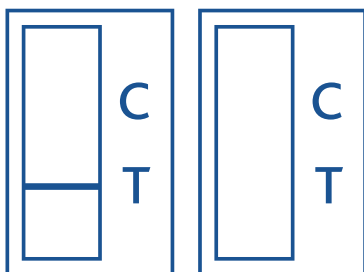

INVALID

Screen result cannot be interpreted

- The presence of 1 line at “T” or no lines at all
- This does not indicate your viral load; it indicates a malfunction of the kit itself
- Repeat the screen with a new swab, test device and nasal sample